ISolate

Catalog No. 99264    12 x 6 mL, 2 x 50 mL, 2 x 100 mL

Ready to Use Density Gradient Medium at Two Concentrations

REFERENCES

Adaniya GK, Jackson KV: *Comparison of Percoll® and ISolate in the Preparation of Semen for use in Assisted Reproductive Technologies*. 53rd Annual Meeting of the American Society for Reproductive Medicine, Cincinnati, Ohio; 0-113, 1997.

Tarchala SM, Fahy MM, Hauserman HM, Kallmann K, Volentine K, Radwanska E, Binor Z, Molo MW, Rawlins RG: *ISolate As a New Method of Sperm Separation*. 53rd Annual Meeting of the American Society for Reproductive Medicine, Cincinnati, Ohio; P-116, 1998.

Miller, KF; Fry KL, Arciaga RL, Falcone T: *Semen preparation for in-vitro fertilization using ISolate results in sperm recovery, fertilization and pregnancy outcomes that are indistinguishable from those obtained with Percoll®*. 14th Embryology, Goteborg, Sweden; P-116, 1998.

Tarchala SM, Volentine KK, Rawlins RG: *A comparison of sperm processing using ISolate and Pure Sperm™*. 14th Annual Meeting of the European Society of Human Reproduction and Embryology, Goteborg, Sweden; R-041, 1998.

REF

LOT

STERILE A

Expiration:  
Year - Month - Day

Caution, consult accompanying documents

Consult instructions for use

Storage Temperature  
2-8°C

Do not sterilize

Do not re-use

Do not use if package is damaged

Manufacturer

U.S. Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

CE Mark

Emergo Europe - Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands

Glossary of Symbols\*:

Catalog Number

Lot Number

Sterilized using aseptic processing techniques (filtration)

Expiration:  
Year - Month - Day

Caution, consult accompanying documents

Consult instructions for use

Storage Temperature  
2-8°C

Do not sterilize

Do not re-use

Do not use if package is damaged

Manufacturer

U.S. Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

CE Mark

Emergo Europe - Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands

\*Symbol Reference - EN ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labeling.

ENGLISH

EU Caution: For Professional Use Only.

INDICATION FOR USE:  
ISolate is intended for assisted reproductive procedures that involve the manipulation of human sperm. ISolate is intended for the separation of the motile fraction of sperm from seminal fluid.

DEVICE DESCRIPTION  
ISolate is a density gradient medium designed to separate the motile fraction of sperm from seminal fluid. As a two layer gradient system, it effectively reduces cellular contaminants such as dead sperm, white blood cells and miscellaneous debris. The resulting sample contains predominantly motile sperm.

COMPOSITION

Salts and Ions

Sodium Chloride

Potassium Chloride

Magnesium Sulfate

Potassium Phosphate

Calcium Chloride

Buffer

Sodium Bicarbonate

HEPES

Energy Substrate

Glucose

Sodium Pyruvate

Sodium Lactate

Other

Colloidal Suspension of Silica

Particles

Note: The quality of the sperm sample should be evaluated and taken into consideration when determining the proper centrifugation speed and time. These should be adjusted according to the individual specimen quality for optimization of the procedure.

6. Using a new sterile pipette, add 2.0 - 3.0 mL of appropriate washing medium such as Sperm Washing Medium (Catalog #9983) or Modified HTF (Catalog #90126) with protein supplementation. Resuspend the isolated pellet. Centrifuge (~200xg) for 8 -10 minutes and remove the supernatant. Repeat this step for a second wash. Discard the supernatant and resuspend the pellet using a suitable volume of appropriate medium. The sample is now ready for analysis.

For additional details on the use of these products, each laboratory should consult its own laboratory procedures and protocols which have been specifically developed and optimized for your individual medical program.

STORAGE INSTRUCTIONS AND STABILITY

Store the unopened containers refrigerated at 2° to 8°C.

Warm to ambient or incubator (37°C) temperature prior to use.

Do not freeze or expose to high temperatures.

ISolate is stable until the expiration date shown on the box and bottle labels when stored as directed.

PRECAUTIONS AND WARNINGS

This device is intended to be used by staff trained in assisted reproductive procedures. These procedures include the intended application for which this device is intended.

The user facility of this device is responsible for maintaining traceability of the product and must comply with national regulations regarding traceability, where applicable.

ISolate will appear opaque. This is normal for this product. Do not use any vial of medium which shows evidence of particulate matter or contamination (nonuniform cloudiness).

ISolate should be tightly capped when used in a CO<sub>2</sub> incubator to avoid pH changes.

Information on known characteristics and technical factors that could pose a risk if the product were to be re-used have not been identified therefore the product is not to be used following the initial use of the container.

In case of damage, do not use. Please discard or return to Manufacturer for replacement.

DIRECTIONS FOR USE

Catalog #99264 - READY TO USE COLLOIDAL SILICA AT TWO CONCENTRATIONS (50% and 90%)

A. Materials Suggested:  
ISolate  
• FUJIFILM Irvine Scientific, Inc. Catalog #99264 (6 mL, 50 mL or 100 mL vial kit)  
Sperm Washing Medium  
• FUJIFILM Irvine Scientific, Inc. Catalog #9983  
Centrifuge  
• FUJIFILM Irvine Scientific, Inc. Catalog #IS-300  
Sterile, disposable conical centrifuge tubes  
Pasteur Pipettes, sterile  
Incubator, 37°C

B. Gradient Preparation:  
No preliminary dilutions are necessary. Each vial of upper and lower layer gradient is ready for use. Continue to GENERAL PROCEDURE section of product insert.

GENERAL PROCEDURE

The following is a general procedure for a two-step gradient separation of motile sperm from semen. The volume and concentration can be modified to meet each laboratory's preference (i.e. mini-gradient, cryopreserved semen, three-layer gradient).

1. Bring all media components to room temperature or 37°C.

2. Using a sterile pipette, transfer 1.5 - 2.0 mL of the "LOWER LAYER" into a sterile, disposable, conical centrifuge tube.

3. Using a new sterile pipette, transfer an equal volume of "UPPER LAYER" on top of the "LOWER LAYER". This is done by contacting the surface of the "LOWER LAYER" at the side of the tube with the tip of the pipette. Carefully dispense the "UPPER LAYER" by spiraling the pipette tip around the circumference of the tube in an upward motion as the level of the "UPPER LAYER" rises.

4. Gently place 1.5 - 2.0 mL of liquefied semen onto the "UPPER LAYER" using a new sterile pipette.

5. Centrifuge for 10-20 minutes at approximately 200 - 300 x g.\* Carefully expose the pellet by either aspirating off the "UPPER" and "LOWER LAYERS", or directly removing the pellet and transferring to a new sterile centrifuge tube.

FUJIFILM Irvine Scientific, Inc.

2511 Daimler Street, Santa Ana, California 92705-5588 USA

Telephone: 1 949 261 7800 • 1 800 437 5706 • Fax: 1 949 261 6522 • [www.irvinesci.com](http://www.irvinesci.com)

© 2023 FUJIFILM Irvine Scientific Inc. All rights reserved. The FUJIFILM Irvine Scientific logo and ISolate are trademarks of FUJIFILM Irvine Scientific, Inc. in various jurisdictions. All other trademarks are the property of their respective owners.

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- FUJIFILM Irvine Scientific, Inc., kataloška št. 9983
- FUJIFILM Irvine Scientific, Inc., kataloška št. IS-300