Freezing Medium - TEST Yolk Buffer (TYB)
with Glycerol and Gentamicin

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Sterilized using aseptic processing techniques (filtration)

Expiration:
Year - Month - Day

Caution, consult accompanying documents

Consult instructions for use

Storage Temperature
below -10 C

Manufacturer

Do not re-use

Do not resterilize

Do not use if package is damaged

U.S. Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

CE Mark

Emergo Europe - Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

*Symbol Reference - **EN ISO 15223-1**, Medical devices – Symbols to be used with medical device labels, labeling.

ENGLISH

EU Caution: For Professional Use Only

INDICATION FOR USE

Freezing Medium is intended for cryopreservation of human semen for use in assisted reproductive procedures.

DEVICE DESCRIPTION

Component

Specification

Formulation: (see references 3 and 4)

Major Components:

TES, Tris, Sodium Citrate and Fructose

Gentamicin Sulfate..... 10 µg/mL

Glycerol..... 12 % (v/v)

Heat-inactivated egg yolk obtained from specific pathogen free (SPF) laying flocks... 20 % (v/v)

COMPOSITION:

Antioxidant

Sodium Citrate

Buffers

TES Buffer

Trizma base

Cryoprotectant

Glycerol

Protein

Chicken Egg Yolk

Energy Substrates

Fructose

Antibiotics

Gentamicin Sulfate

Water

WFI Quality

QUALITY ASSURANCE

Freezing Medium is membrane filtered and aseptically processed according to manufacturing procedures which have been validated to meet a sterility assurance level (SAL) of 10⁻³.

Each lot of Freezing Medium is tested for:

Endotoxin by Limulus Amebocyte Lysate

(LAL) methodology

Sperm Cryo Survival Assay

Sterility by the current USP Sterility Test <71>

All results are reported on a lot specific Certificate of Analysis which is available upon request.

DIRECTIONS FOR USE

- Semen is collected following a two to three day abstinence period.
- Sample is allowed to liquefy at 37 C for 30 minutes.
- One vial of medium is thawed and brought to 37 C.
- The liquefied sample is transferred to a sterile, 15 mL, conical centrifuge tube, the volume determined and medium added dropwise until a 1:1 sample:medium ratio is achieved.

Note: Samples displaying high viscosity may require the additional step of repeated pipetting or passage through an 18 gauge needle to ensure thorough mixing.

- The sample-medium mixture is placed in a beaker or other suitable container of 37 C water.
- The container is refrigerated at 2 C to 5 C to allow a slow cooling of the mixture (0.5 C/minute). After 90 minutes, the sample is ready to freeze in straws, vials or pellets using traditional procedures.

Note: Steps 1-5 can be performed at room temperature.

For additional details on the use of these products, each laboratory should consult its own laboratory procedures and protocols which have been specifically developed and optimized for your individual medical program.

STORAGE INSTRUCTIONS AND STABILITY

Store the unopened bottles at -10 C or colder. When stored as directed, Freezing Medium is stable until the expiration date shown on the bottle or vial label.

Do not expose medium to repeated freeze thaw cycles. If smaller aliquots are desired, thaw the product, aliquot into sterile labeled containers and freeze until time of use.

PRECAUTIONS AND WARNINGS

This device is intended to be used by staff trained in assisted reproductive procedures. These procedures include the intended application for which this device is intended.

The user facility of this device is responsible for maintaining traceability of the product and must comply with national regulations regarding traceability, where applicable.

Freezing Medium will appear opaque. Do not use any bottle of medium which shows evidence of particulate matter or is not yellow in color.

To avoid problems with contamination, handle using aseptic techniques and discard any excess medium that remains in the bottle or vial after the procedure is completed.

Information on known characteristics and technical factors that could pose a risk if the product were to be re-used have not been identified therefore the product is not to be used following the initial use of the container.

CONTRAINDICATION

Product contains Gentamicin Sulfate and egg yolk protein. Appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic or egg yolk protein.

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